

**"I know my value."
You too can be more involved in
your anticoagulation therapy**

[Find out more](#)
[Training & Support](#) > [CoaguChekIN Range Firmware Update](#)

CoaguChek INRange / Vantus (US only) Firmware Update

The following steps describe the firmware update process for the CoaguChek INRange / Vantus (US only) system via USB.

Update Instructions - PDF

[How to update my CoaguChek INRange / CoaguChek Vantus \(US only\)](#)

Note: the firmware version 01.05.00 is compatible with both devices **CoaguChek INRange and CoaguChek Vantus (US only)**

Firmware files

30-May-2018

[Update 01.04.14-YT](#)
[Release Notes](#)

29-Aug-2019

[Update 01.05.00-YT](#)
[Release Notes](#)

Prerequisites

- ◆ CoaguChek INRange/Vantus (US only) meter
- ◆ USB Type-A to Micro-USB cable (included in package)
- ◆ Full AAA Batteries (Alkaline)
- ◆ Windows or Mac Computer
- ◆ Firmware file provided on this page (01.05.00-YT.znu)



Prerequis

Note: Certain countries have to register the software separately and therefore the software is only available for download once the registration is performed.

All CE Countries of the European Union are able to upload this Software immediately.

All the other countries can contact the Author of this Support Notification or your Roche Service organization, once the software is registered, in order for the software to be released for download

Step 1 - Prepare Meter for Firmware Update



- ◆ insert Batteries
- ◆ Power on CoaguChek INRange/Vantus Meter
press on/off button



- ◆ Go to Menu "Other"
press arrow down button
- ◆ Select Menu "Other"
press enter button






- ◆ Go to "Firmware Update"
press arrow down button
- ◆ Select "Firmware Update"
press enter button

Step 2 - Connect Meter with Computer



- ◆ Make sure computer is powered on and user is logged in
- ◆ Connect Micro-USB cable with meter
- ◆ Connect USB cable to computer
- ◆ Display on meter will show "For Firmware Update copy file to CoaguChek INRange/Vantus-Drive"

Step 3 - Copy Update File to Meter

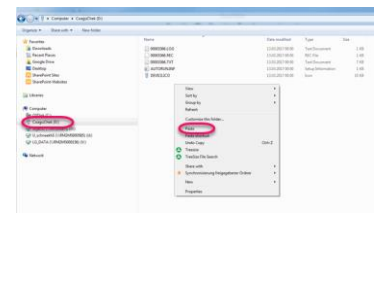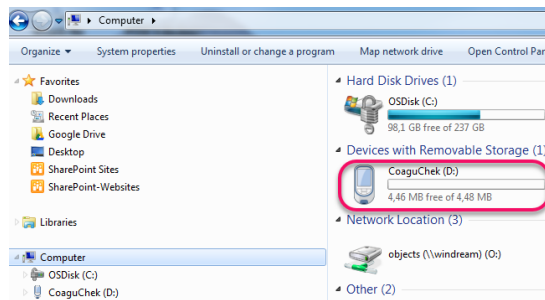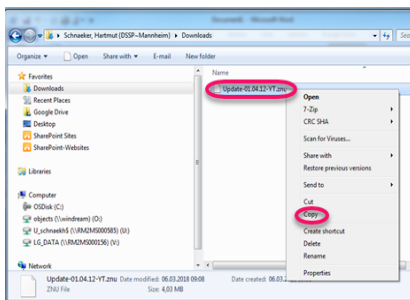

C or CMD + C (Mac)

- ◆ Download Update file to computer (if not already done)
- ◆ Locate the file in Windows Explorer or Finder (Mac)
- ◆ Select file, e.g. "Update01.05.00-YT.znu"
- ◆ Right click on file "Update01.05.00-YT.znu" and select "Copy"; alternatively, press CTRL +

Locate CoaguChek INRange/Vantus meter in your Windows Explorer or Finder (Mac) – It will be listed as a local drive - like a USB memory stick

Select CoaguChek INRange/Vantus meter in your Windows Explorer or Finder (Mac)



Right click inside CoaguChek INRange/Vantus and select "Paste"; alternatively, press CTRL + V or CMD + V (Mac)





- ◆ update process is started
- ◆ CoaguChek INRange / Vantus will execute the update automatically within several minutes
- ◆ Displays will automatically change during the update process

Step 4 - Verify Firmware Version



- ◆ Wait until meter automatically powers off after the update process
- ◆ Turn on the meter
Press on/off button



- ◆ Go to Menu "Other"
press arrow down button
- ◆ Select Menu "Other"
press enter button



- ◆ Select "About"
press enter button
- ◆ Check for Firmware "Update01.0 5.00-YT" on the display

If Firmware listed on display is different than "01.05.00 YT", the update is not complete. Please try again to update the CoaguChek INRange / Vantus meter by repeating the steps described. If the update still does not work, please contact Roche Diagnostics.