

New firmware version 01.04.12 for your CoaguChek INRange meter

Date: March 2018

Rev. 1.0

The new CoaguChek INRange firmware version 01.04.12 includes the following changes:

- The target range setup workflow has been optimized for a more intuitive handling. Additionally the default target range has been altered to 2.0 – 3.0 INR (before: 1.5 – 2.5 INR).
- When downloading the reports to a computer:
 - the user may now select up to 365 days as trend graph resolution (before: up to 180 days).
 - the RELEASE.TXT file is removed
 - the order of date/time elements in the Arabic language was fixed

Should you have any question about the CoaguChek INRange firmware version 01.04.12, please contact your Roche Service organization.

Best Regards